




MANUAL STERILIZATION USING AMPOULES · INSTRUCTIONS OF USE

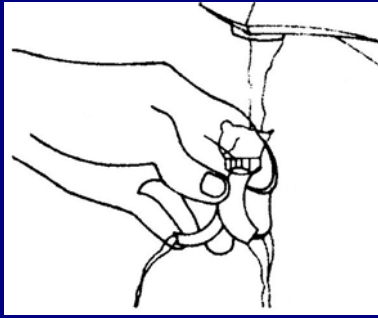


1. Wash the elements to be sterilized with water and soap.

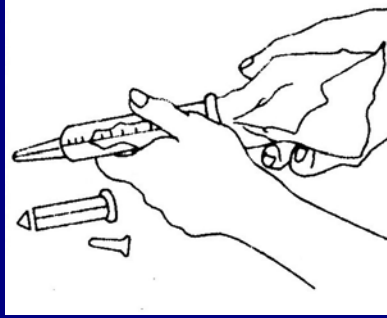


2. Then dry them carefully, before wrapping them individually.

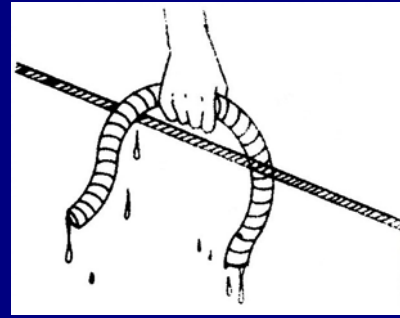


3. Hang up the long lengths of plastic or rubber tubing so that they drip all the water that is inside them.

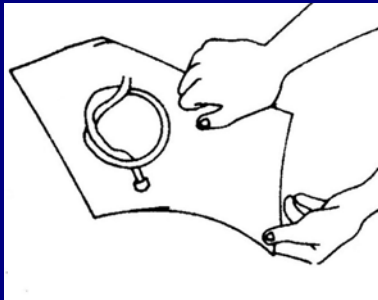


4. You must wrap all items individually in cloth, paper or polyethylene.

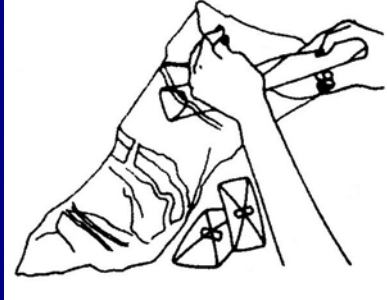


5. Place those wrapped items inside the retaining bag.

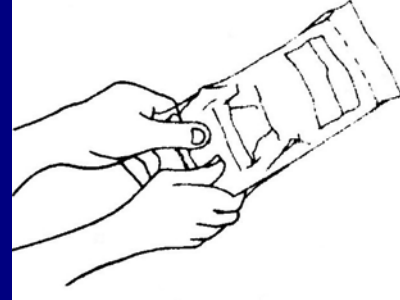


6. Take one BIOLENE® ampoule without opening the plastic bag in which it is wrapped. Rather, roll it out. Grasp the ampoule through the bag and protective shield and snap the top of the ampoule.

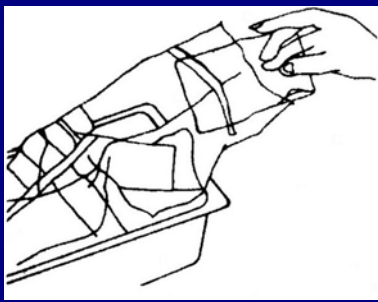


7. Place the ampoule and the wrapped items inside the retaining bag.

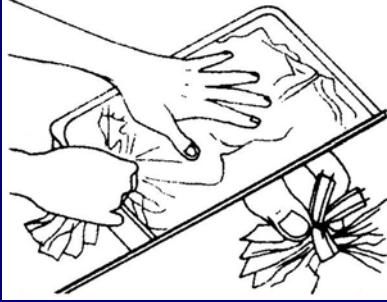


8. Remove the air from the retaining bag, twist it, shut and hold the seal with one of the twist seal closures provided.

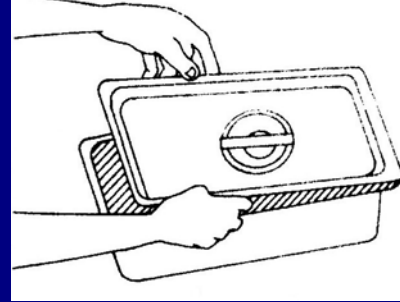

9. Place the kit in the container and lock the lid in place. Mark the top of the container with the date and hour that sterilization was begun.

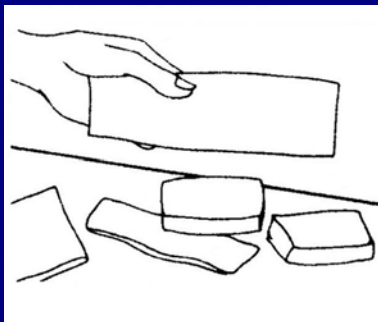

10. When twelve hours have passed, remove the sterilized materials

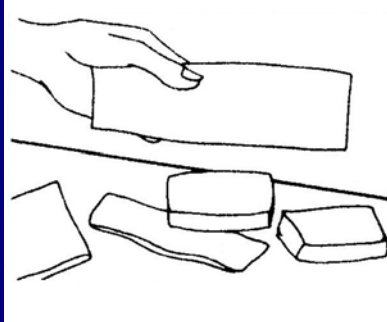

11. Place them wrapped for 24 hours so that they ventilate. The elements of high porosity or absorption to be used on the human body must be ventilate during 72 hours before use.



12. The retaining bag and the bag of the ampoule must be disposed of in ordinary rubbish.