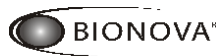
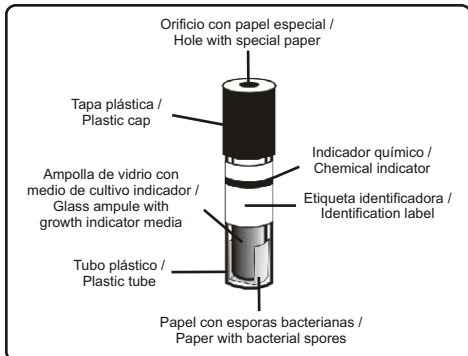
Indicadores Biológicos Biological Indicators

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Composition

Each tube contains a population of *Geobacillus stearothermophilus* ATCC 7953 spores soaked on a strip of paper. It also has a growth indicator media of purple color contained in the glass ampule.

Product description

Bionova BT20 biological indicator is specifically designed for the monitoring of vacuum assisted steam sterilization processes at 121°-134°C and for steam sterilization cycle at 121°-123°C in a gravity displacement air.

If sterilization process was not successful, the indicator media will change from purple to yellow after incubation at 56±2°C, thus indicating the presence of live *Geobacillus stearothermophilus* spores.

If the sterilization process was successful, the indicator media will remain purple after incubation. The final readout should be made after 24 hours of incubation at 56±2°C.

Warning!

Do not use BT20 biological indicator for monitoring gravity steam sterilization cycle at 132°-135°C, or ethylene oxide, dry heat, chemical vapor, radiation or other sterilization processes.

Do not re-use the biological indicators.

Important considerations

You should use at least two BT20 biological indicators to monitor each steam sterilization cycle.

Storage

Store it in a dark place and temperatures between 15-30°C, 35-60% relative humidity.

Do not freeze.

Do not store these biological indicators near sterilizing agents or other chemical products.

Directions for use

1. Identify the Bionova BT20 indicator by writing the sterilizer number (in case it has more than one), load number, and processing date on the indicator label.

2. Pack the biological indicator along with materials to be sterilized in an appropriated package according to recommended sterilization practices. Place this package in those areas which a priori you consider most inaccessible for sterilizing agent (high temperature steam). Generally a problematic area is the center of the load and near the door.

3. Sterilize as usual.

4. After the sterilization process has finished, open the sterilizer door, wait for 5 minutes and remove biological indicator from package. **Precaution!**: Wear safety glasses and gloves when removing the BT20 biological indicator from the sterilized package. **Warning!**: Do not crush or handle the biological indicator excessively, since this might cause the glass ampule to burst.

5. Allow the biological indicator to cool until it is under room temperature.

6. Check the chemical indicator on the label of biological indicator. A color change from pink to brown confirms that the biological indicator has been exposed to steam at 121°C or more.

Important: this color change does not indicate that the process was sufficient to achieve sterility. If the chemical indicator is unchanged, check the sterilization process.

7. Crush the glass ampule contained in the biological indicator and incubate at 56±2°C.

Important: Use a non-sterilized biological indicator as positive control each time a processed biological indicator is incubated. The positive control ensure that correct incubation conditions were met.

8. Incubate the processed biological indicator and the indicator used as positive control for a maximum of 24 hours at 56±2°C. Reading should be made at convenient intervals of 10 hours.

A color change from purple to yellow of the growth indicator media means a sterilization process failure has occurred. If after 24 hours there is no color change in the processed indicators, a final negative result is made (the sterilization process was acceptable). The positive control indicator should show a purple to yellow color change for the result to be valid.

Record the positive ones and discard them immediately as it is shown below.

Warning!: Do not use the sterilizer until the biological indicator test results are negative (process indicator reads purple).

Disposal

Dispose of used biological indicators according to your country's healthcare and safety regulations. The positive biological indicator can be autoclaved at 121°C for at least 15 minutes, or at 132°C for 10 minutes in a gravity displacement steam sterilizer, or at 132°C for 4 minutes in a vacuum assisted steam sterilizer.

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