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## 56°C

### Product description

The Bionova® IC20 incubator is designed for Bionova® biological indicators incubation. The incubator temperature is pre-set at  $56 \pm 2^\circ\text{C}$ . **Important:** this temperature can not be modified.

If the sterilization process was not successful, the growth medium contained into the indicator will turn to yellow as is indicated in the biological indicator directions for use.

If the sterilization process was successful, the indicator media color will remain unchanged.

### Operating conditions

The Bionova® IC20 incubator works at environmental temperature of  $15-35^\circ\text{C}$ , a relative humidity of 35-85%, and a voltage of 110 or 220 volts depending on it is indicated in the apparatus.

### Symbols

35-85% Operating environmental relative humidity.

$15-35^\circ\text{C}$  Operating environmental temperature.

Caution.

Caution: risk of electric shock.

Caution: hot surface.

Attention: read usage and operating instructions.

### Warning!

Do not pour any liquid inside.

Do not immerse in any liquid.

Use indoors only.

Disconnect the power cord before cleaning.

Do not use abrasive or corrosive cleaners or disinfectants.

The incubator must be sent to the manufacturer to be repaired.

Make sure that the incubator is connected to a properly rated power cord.

### Directions for use

1. Put the biological indicator into the metal incubator fastener as is shown in the figure A. **Important:** Check that the incubation temperature is correct.

2. Break the ampoule pressing backwards as is shown in figure B.

3. Remove the biological indicator from metal support grabbing it by the lid. Next, spread the liquid medium hitting the tube base with the forefinger, figures C and D.

4. Get back the biological indicator to the incubator fastener, figure E.

5. Incubate at  $56 \pm 2^\circ\text{C}$ .

**IMPORTANT!** Use a biological indicator which has not been exposed to sterilization process as positive control each time a processed indicator is incubated. The positive control ensure that correct incubation conditions were met.

Incubate the processed biological indicator and the indicator used as positive control at  $56 \pm 2^\circ\text{C}$  for as long as specified on the biological indicator's directions for use. Reading should be made at convenient intervals of 10 hours.

The color change of the growth indicator medium to yellow means a failure in sterilization process. If after the incubation time (see biological indicator's direction for use) there is no color change in the processed indicators, a final negative result is made (the sterilization process was acceptable).

**Important!** The color of biological indicator growth media used as positive control has to change to yellow during incubation for results to be valid.

Record the positive ones and discard them immediately as is shown in the biological indicator's directions for use.






**Warning!** Do not re-use the sterilizer until the biological indicator test result is negative (the processed indicator color remains unchanged).

### Warranty

Terragene S.R.L warrants the material components quality of this product and its manufacturing process. If material or manufacturing defects are detected within warranty period, the unique Terragene's S.R.L. obligation will be the product repairing or the substitution.

### Limitation of liability:

Terragene S.R.L is not liable for any loss and damage arising from the unsuitable use of this equipment, negligence or strict user responsibility.

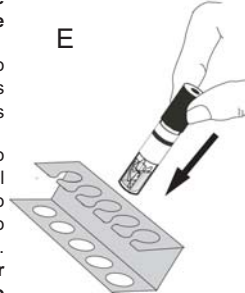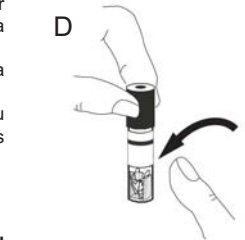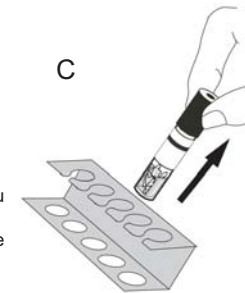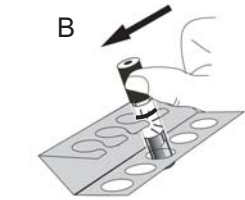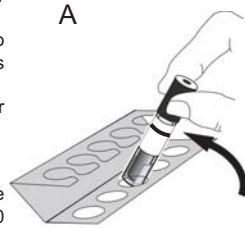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